

A glowing sphere of musical notes and symbols against a dark background. The sphere is composed of numerous small, glowing musical notes, clefs, and symbols, creating a dense, textured surface. The lighting is warm and golden, emanating from the center of the sphere and casting a soft glow on the surrounding dark space. The overall effect is one of vibrant energy and musical harmony.

Listen . . . You're made of Music

StateSong® Translates Biometric Data Into Music

What If . . .

We could create music from our inner biological states ...
Meditation, Focus, 2nd Wind, Sleep, Dance, Happiness?

What if this music can be used to SYNC us into those states?



CalmSong



FocusSong



2ndWindSong



CreateSong

With StateSong, It's Possible ...

StateSong lets you create and record Music from your E4, giving you songs made from your internal states such as sleep, rest, meditation, focus, running, and more!; then you can then use your StateSongs to reach those states.



This guide will help you install and use the StateSong app.

Sample StateSongs from E4

Below are links to 5 StateSongs created from E4 data. The titles reflect the state of the user during the recording. Click on the [links](#) below to play each StateSong.

These 1st generation StateSongs are conversions of E4 data to music with no intervention other than assigning a sound to each biometric data stream. They are musical translations of your internal states. We call it 'Endogenous Music', or 'Music from the Inside'. With coming improvements, combined with AI, we foresee StateSong emulating full bands, orchestras & all genres of music.



[Meditation](#)



[Focus](#)



[Gardening](#)



[Ponder](#)



[Stressed](#)

Disclaimers

Please note, this is an 'beta' version of the StateSong app. While this beta is openly downloadable on the Google Play Store, is currently only usable with Empatica E4 devices connected to mobile phones using Android version 9 or above. There is no charge nor limits on the use of this app.

Also, while **no personal data is, or ever will be retained by MediMuse**, but we do collect web application errors and messages. This allows us to make functional and interface improvements to the StateSong application.

Your feedback is key to continued StateSong development. Send comments/suggestions to peterjslack@medimuse.dev







Installing and Permissions

To install the app, go to the Google Play Store on your Android device. Search for StateSong. Look for this logo. Click on the logo to go to the install page



StateSong by Medimuse (TM)
Peter Slack • Music & Audio
📄 Installed

Click on the green Install button. Once installed, Open. Allow Access to Location, Audio, Photos & Bluetooth.

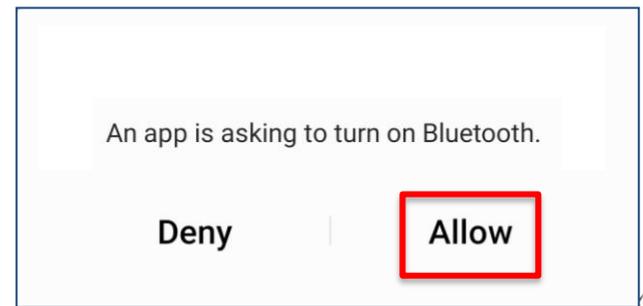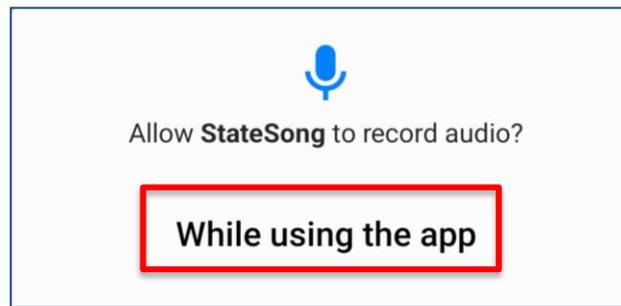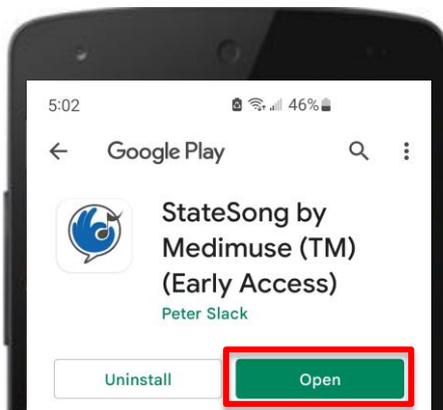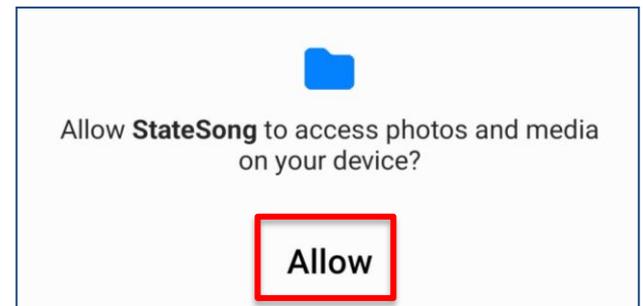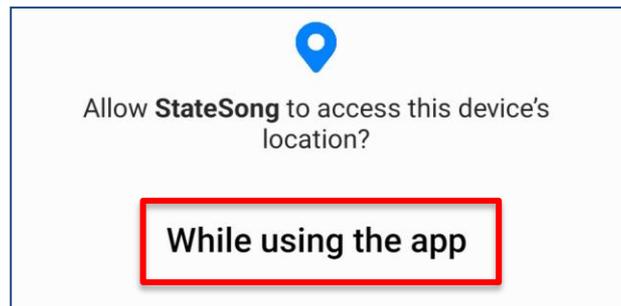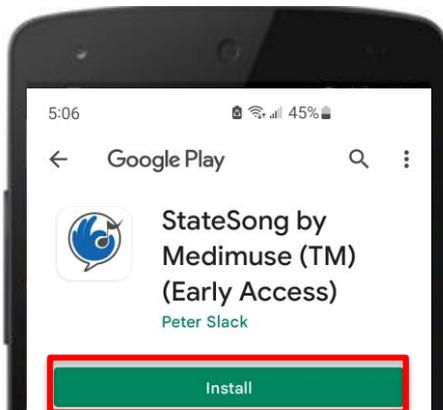


Setting Up The StateSong App

With the app open, click 3 dots  on the top right corner.






Select 'Settings' from menu.

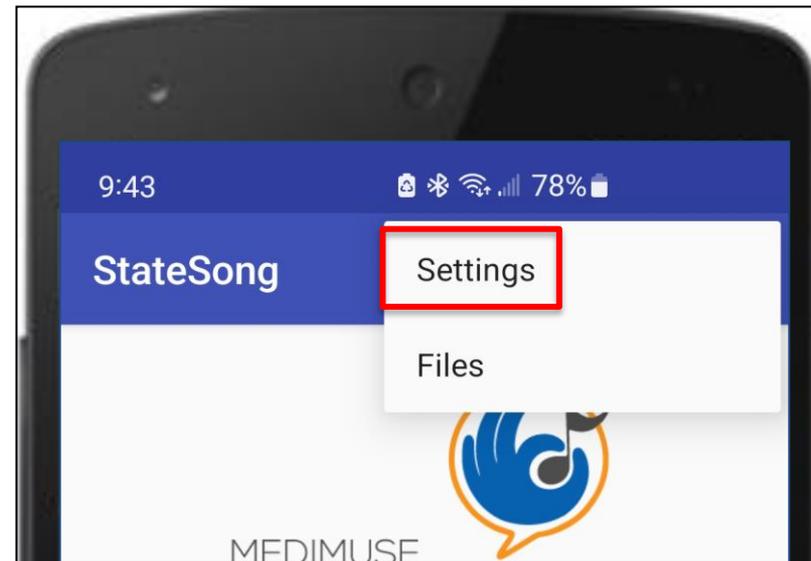



Select Synthesizer / Sequencer

In Settings, select the 'MediMuseStateSong Sequencer'. Tap either of the arrows to go back to the main view.

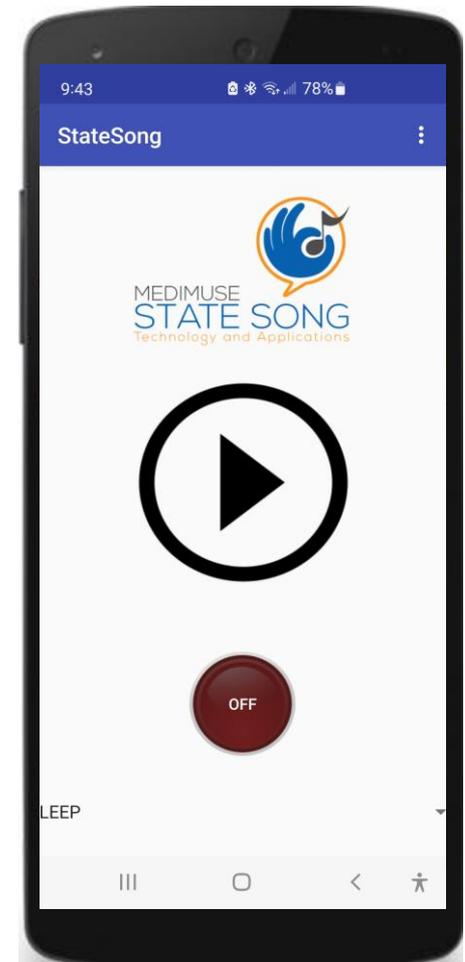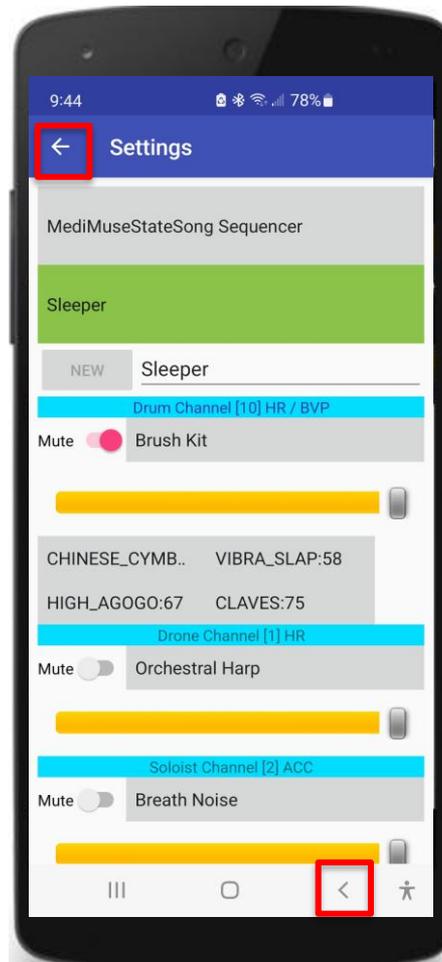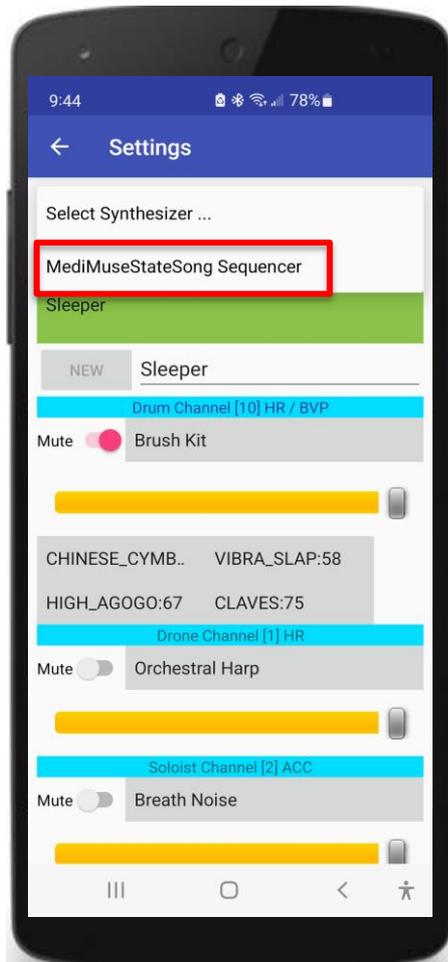

Connecting StateSong to E4



Turn on E4. Then push the Arrow in the center of the app.
When E4 light turns solid blue, the E4 & app are connected.

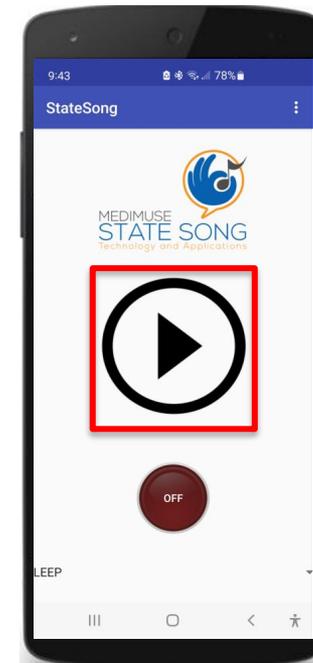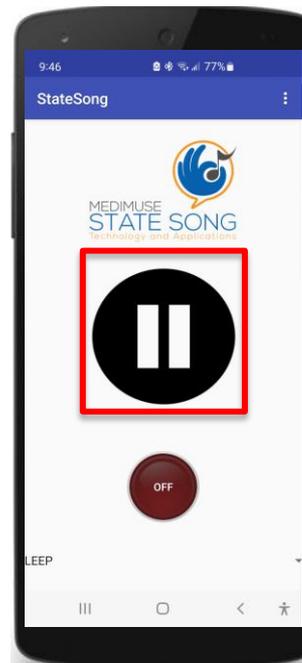

If you don't music sounds after 15 sec, tap pause, then tap play again. (It can take a minute to begin converting data)

Select a 'Default' State

Tap the word 'SLEEP' or the lower right 'Arrow'. You'll see: SLEEP, REST, MEDITATION, FOCUS, ACTION and EXCERSIZE

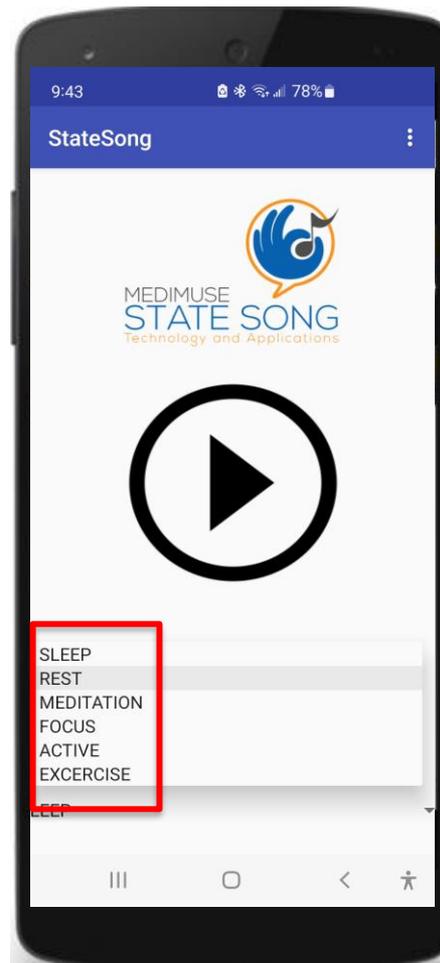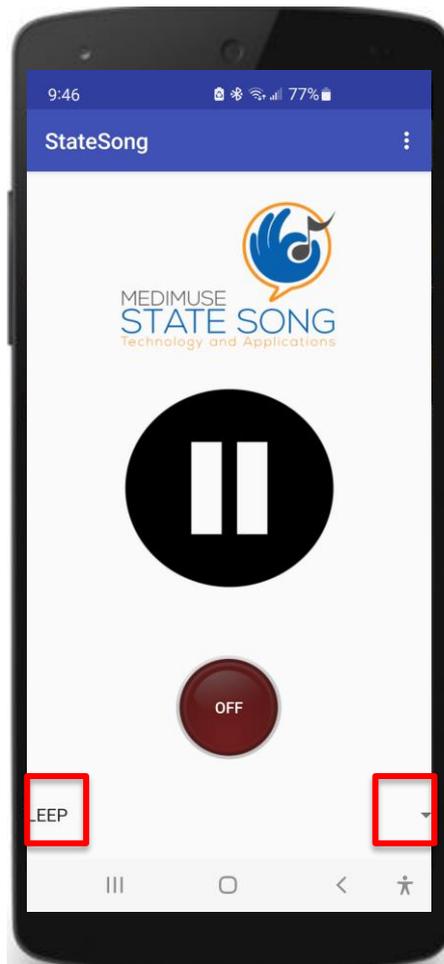


These are Default, or Primary 'State Settings'.

Select the state from which you want to create a StateSong.

As your body attempts to reach the targeted state, StateSong will turn your E4 data into music from that state.

Changing Default Settings


To change and save default state sounds settings, tap  to, see Drum, Drone, Solo, Bass, Harmony, and Nature Channels

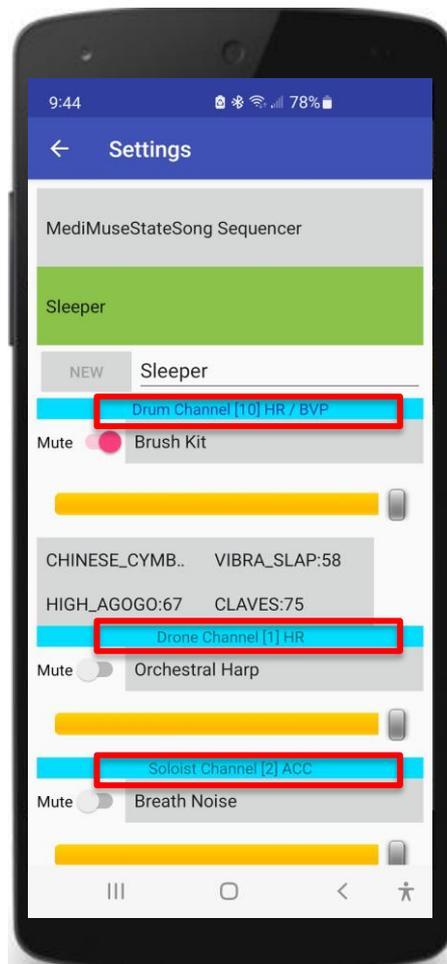

Channels convert data from the E4's biometric sensors into sounds.

You can change sounds by tapping the gray bar below each blue bar.


There are 4 drums in the drum channel. Select a kit, or change a single drum (i.e. snare > conga).

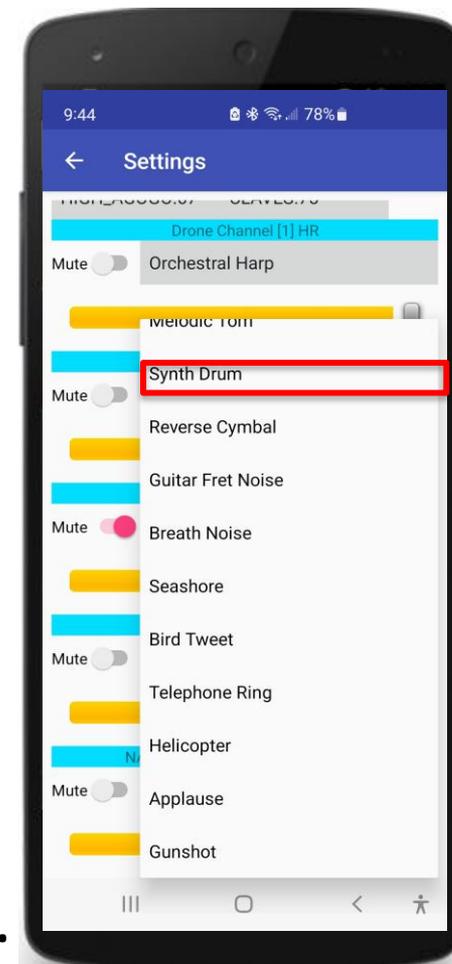


Please note: the Harmony channel is currently not functional, but will be connected to GSR biometric in the next version

Saving State Sound Settings

You can rename a State after changing channel sounds; for example, change sleep to sleeper, or exercise to jogging.



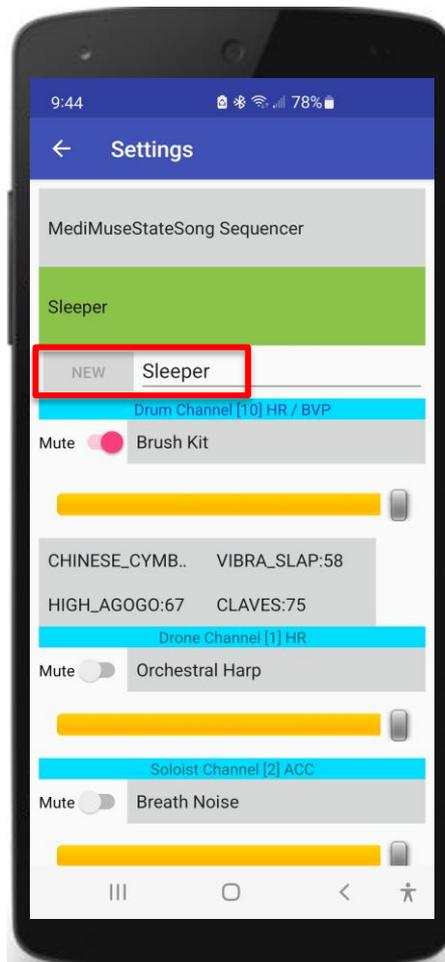


The current name shows in a green bar below the Sequencer. Once sounds are reset, type in a new name, tap NEW. Note: you can adjust volume or fully mute a channel. New setting names show when you tap the green pull down bar. Tap arrow to return to Main Page

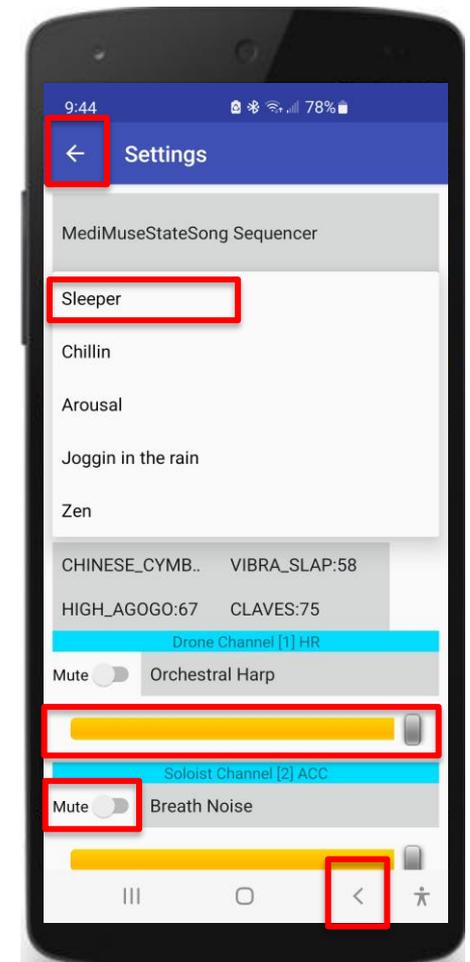


Recording a StateSongs

To record, touch the 'Off' button and it will turn to 'On'. Tap 'On' to stop recording. Tap  to end the session

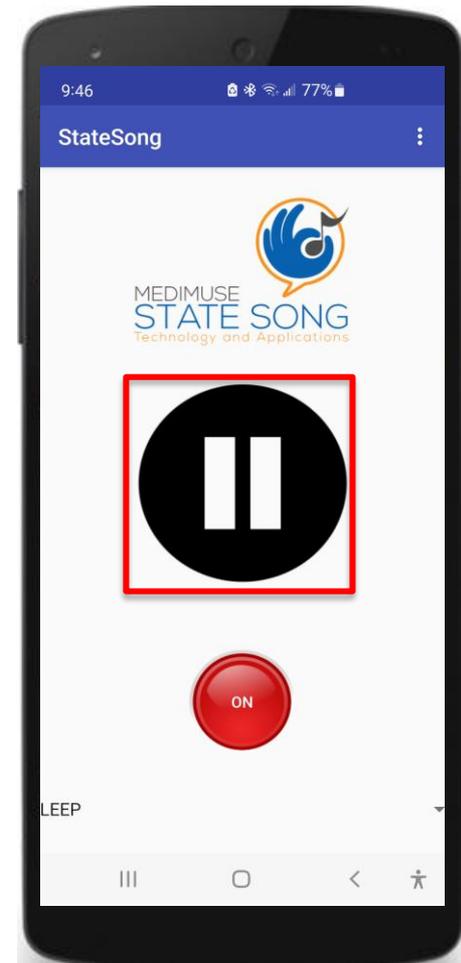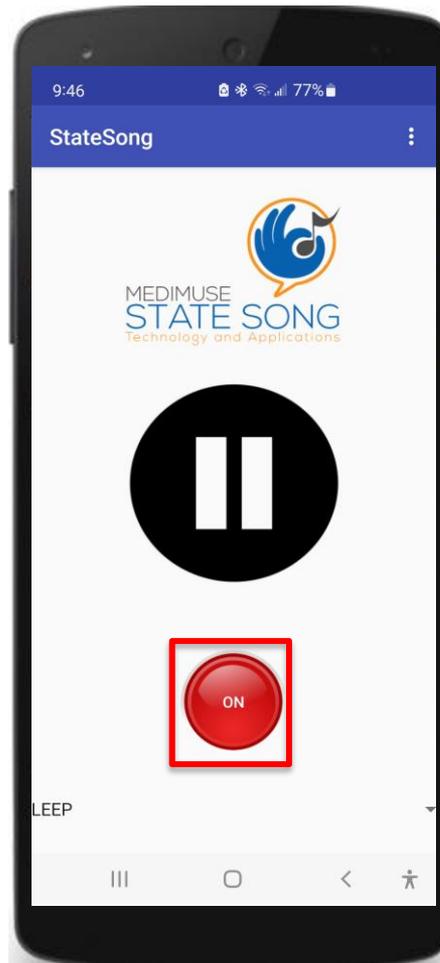
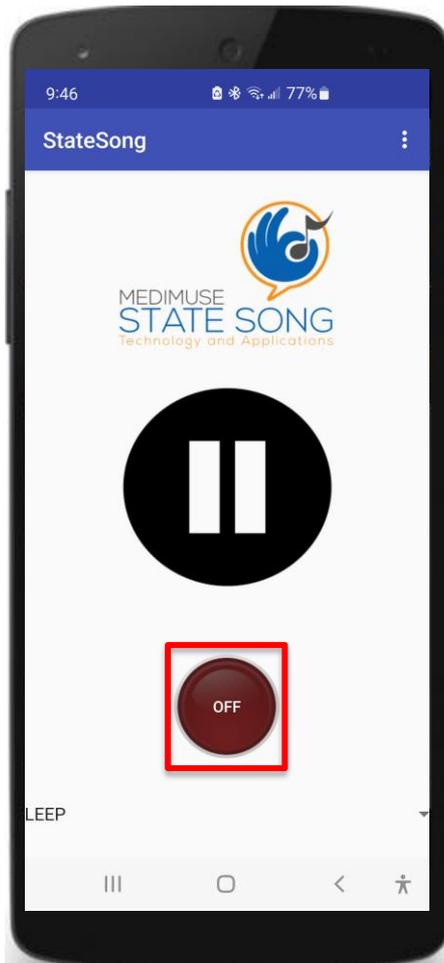


Find & Play StateSongs Files

The Files page is where all your StateSongs are saved.

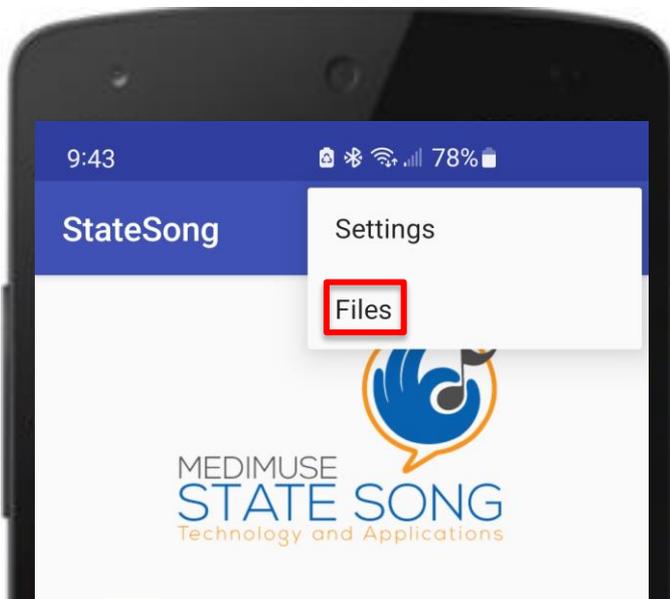


Tap  in the top right corner of the Main Page to access Files option. From this view will be able to see and share all your saved StateSongs as well as your Settings (configs)

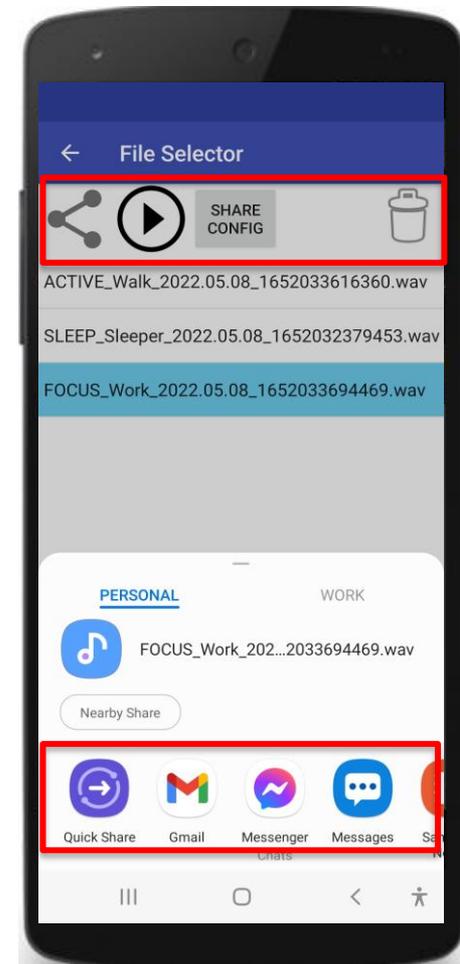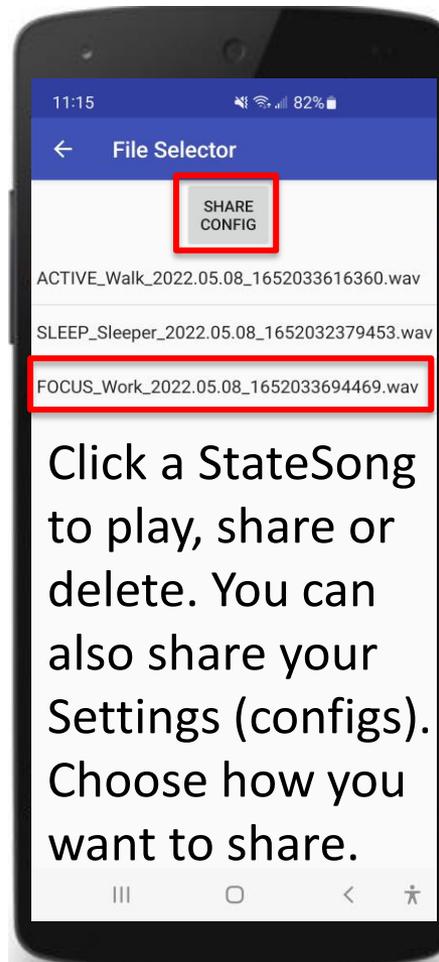


Background Running

StateSong runs in the background as you do other things...

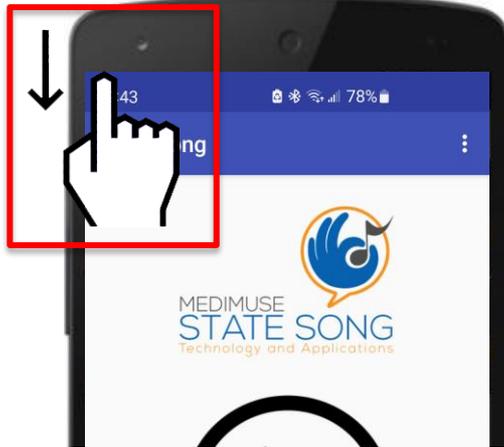

To stop background running, swipe down from top of the screen to show notifications. StateSong Player will appear with a logo and an arrow at far right. Click on arrow to reveal PAUSE, PLAY, and KILL. Click 'KILL' to stop the app from running.

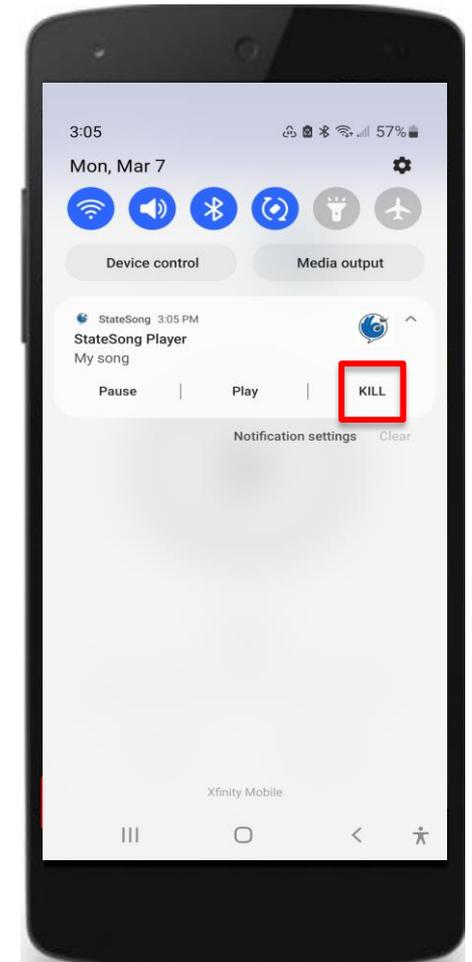

Your feedback is important. Send any comments or suggestions to peterjslack@medimuse.dev

